



EMODIAL

BETAFIX[®]

Fistula Bandage



EMODIAL

Via Finati, 4/A - Ferrara - ITALY

Tel. +39 0532 978198 - Fax +39 0532 978414

www.emodial.com - info@emodial.com

CE
0373

Ref.GB002 - Rev. 07/2019



DESCRIPTION

Elastic fistula bandage provided with a two layer highly absorbent pad and adhesive for a better tightness. Also suitable for patients who require intravenous treatment and blood transfusion.

INTENDED USE

Post dialysis dressing to stop the bleeding after the removal of the fistula needles.

PURPOSE

Designed to stop bleeding and to reduce the hemorrhagic risk, thanks to the compression on the needle holes, also working as a barrier against the risk of infections. The non-adherent, biocompatible layer on the pad doesn't stick to the skin **avoiding the clot removal and the risk of bleeding.**

The cotton bandage avoids the sensitization due to the glue of adhesive.



DIRECTIONS FOR USE



1 Unroll the first part of the bandage containing the pad.



2 Put the pad over the needle site.



3 Tighten bandage sufficiently to stop bleeding.



4 Remove the protective film and fix the bandage with the adhesive.

REFERENCES

Data and technical specifications can be changed without notice.

| CODE | BANDAGE (cm) | | PAD (cm) | PCS / PACK | PACKS / BOX |
|---------|--------------|---|-----------|------------|-------------|
| | W | L | | | |
| S0014 | 50 ± 10% | 6 | 3.5 x 4.5 | 2 | 300 |
| S0029 | 50 ± 10% | 6 | ø 2.6 | 1 | 500 |
| S0029/A | 55 ± 10% | 6 | ø 2.6 | 1 | 400 |
| S0030 | 50 ± 10% | 6 | ø 2.6 | 2 | 300 |
| S0030/A | 55 ± 10% | 6 | ø 2.6 | 2 | 300 |
| S0030/R | 25 ± 10% | 6 | ø 2.6 | 2 | 300 |
| S0041 | 50 ± 10% | 6 | ø 3.5 | 2 | 300 |

CLASSIFICATION • CLASS I MEDICAL DEVICE
 BIOCOMPATIBLE • ACCORDING TO UNI EN ISO 10993 STANDARDS
 STERILIZATION • GAMMA RAYS
 EXPIRATION • 5 YEARS

  • SINGLE-USE